

# NON-TOXIC

# NON-FLAMMABLE

# NO RESIDUE

Code: SOU01014  
**BIO LUBE ADVANCED DENTAL  
HANDPIECE CLEANER**



CAUSES SKIN IRRITATION. CAUSES  
SERIOUS EYE IRRITATION. CONTENTS  
UNDER PRESSURE. KEEP OUT OF  
REACH OF CHILDREN.

### Precautions:

Wash thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. For specific treatment see section 4 of SDS. Avoid release to the environment. Dispose of contents/ container to disposal recycling center. Under RCRA it is the responsibility of the user of the product to determine at the time of disposal whether the product meets RCRA criteria for hazardous waste. Waste management should be in full compliance with federal, state and local laws. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Read label before use.

### First Aid:

IF ON SKIN: Wash with plenty of water. If skin irritation occurs: Get medical advice/attention. Take off contaminated clothing and wash it before reuse.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. Emergency Phone 1-800-424-9300, Chemtrec US, International 1-703527-3887

Info Phone 1-800-880-0240

SOUTHLAND DISTRIBUTION & SALES  
4003 Enterprise Court  
Augusta, GA 30907  
1-800-880-0240  
www.sdssouthland.com



# Bio Lube™

## Advanced Dental Handpiece Cleaner

*Evaporates Oil Buildup  
Removes Blood and Saliva  
Leaves No Residue  
Dries Instantly*



**ENVIRONMENTALLY FRIENDLY**  
**Non-Flammable • Non-Toxic**

Read carefully other precautions on side panel.

NET WT. 7 OZ. (340G)

### DIRECTIONS:

#### Step 1 -Clean


1. Hold Handpiece in a towel, flush drive airline with Bio Lube Advanced Handpiece Cleaner for 2 to 3 seconds.
2. Repeat step 1 until no debris is visible, insuring a clean turbine.

#### Step 2 - Lubricate

1. Spray Bio Lube Lubricant 2 to 3 sec. bursts, or apply 3 to 4 drops
2. Purge Handpiece: run 20 to 30 seconds to expel excess oil.
3. Wipe excess lubricant from Handpiece.
4. Bag and Sterilize Handpiece.
5. Remove from Sterilizer.

### NO NEED TO LUBRICATE AFTER STERILIZATION

\*Use the universal red straw or manufacturer's attachments.

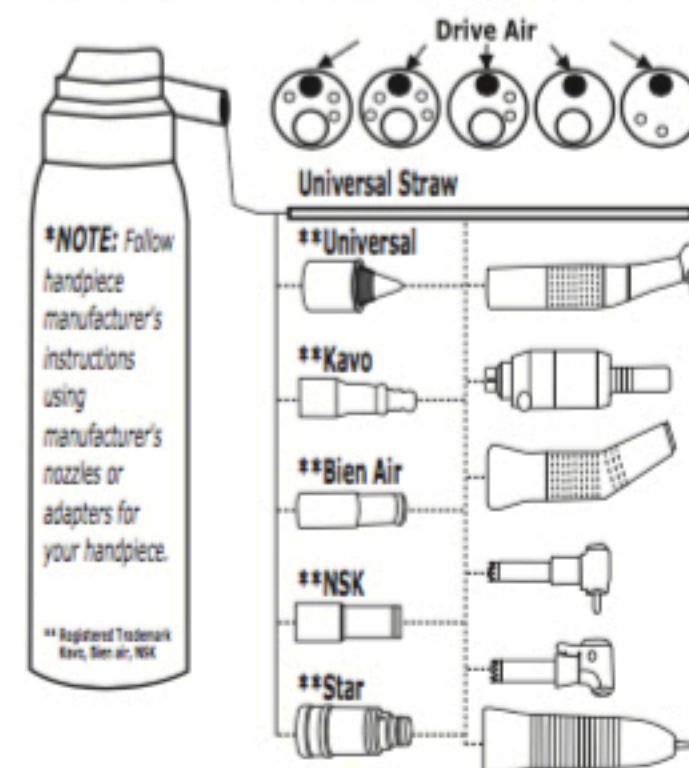

Bio Lube™ Advanced Cleaner is recommended for all Electric Attachments, Highspeed Handpieces, Air Motors and Scalers that require lubrication. Not for Motors with sealed bearings.

REORDER  
NUMBER **BIO-LC-16**