

INSTRUCTIONS

CLEAN - LUBE - PURGE - BAG - STERALIZE

For non-aerosol application, add four drops into Drive Air.

TO CLEAN



1) External Clean-
ing:Scrub the exterior
of each handpiece with
mild detergent and water. Dry.

Internal Cleaning: Holding handpiece in white towel, flush air drive port with Bio Lube Advanced Cleaner using 2-3 second bursts removing water, old oil, and protein build-up. Use your manufacturers' lube adapters to fit quick connect handpieces (**B1**).

A2- Drive Air Port Illustration

