

NON-TOXIC

Code: **SOU01014**
**BIO LUBE ADVANCED DENTAL
HANDPIECE CLEANER**



WARNING

Hazardous Statements

Causes skin irritation. Causes serious eye irritation. Contents under Pressure. Keep out of reach of children. Read carefully other precautions on side panel.

Supplementary Information

Contains: 1,2-DICHLOROETHYLENE

Precautionary Statements:

Wash thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. For specific treatment see section 4 of SDS. Avoid release to the environment. Dispose of contents/ container to disposal recycling center. Under RCRA it is the responsibility of the user of the product to determine at the time of disposal whether the product meets RCRA criteria for hazardous waste. Waste management should be in full compliance with federal, state and local laws. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Read label before use.

First Aid:

IF ON SKIN: Wash with plenty of water. If skin irritation occurs: Get medical advice/attention. Take off contaminated clothing and wash it before reuse.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. Emergency Phone 1-800-424-9300, Chemtrec US, International 1-703527-3887

Info Phone 1-800-880-0240

SOUTHLAND DISTRIBUTION & SALES

4003 Enterprise Court

Augusta, GA 30907

1-800-880-0240

www.sdsouthland.com



REORDER
NUMBER

BIO-LC-16

NON-FLAMMABLE

Bio Lube™

Advanced Dental Handpiece Cleaner

*Evaporates Oil Buildup
Removes Blood and Saliva
Leaves No Residue
Dries Instantly*



ENVIRONMENTALLY FRIENDLY
Non-Flammable • Non-Toxic

Read Safety Warnings on Side Label.

NET WT. 12 OZ. (340G)

NO RESIDUE

DIRECTIONS:

Step 1 - Clean


1. Hold Handpiece in a towel, flush drive airline with Bio Lube Advanced Handpiece Cleaner for 2 to 3 seconds.
2. Repeat step 1 until no debris is visible, insuring a clean turbine.

Step 2 - Lubricate

1. Spray Bio Lube Lubricant 2 to 3 sec. bursts, or apply 3 to 4 drops
2. Purge Handpiece: run 20 to 30 seconds to expel excess oil.
3. Wipe excess lubricant from Handpiece.
4. Bag and Sterilize Handpiece.
5. Remove from Sterilizer.

NO NEED TO LUBRICATE AFTER STERILIZATION

***Use the universal red straw or manufacturer's attachments.**

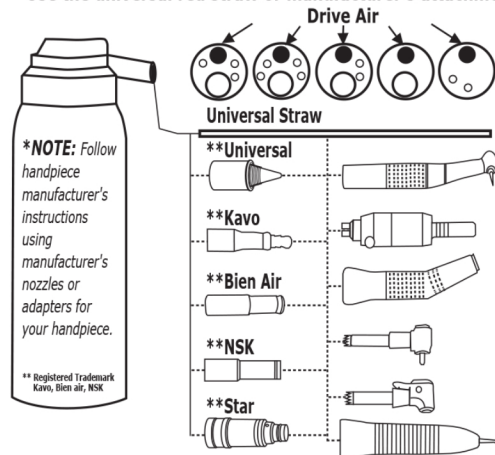

Bio Lube™ Advanced Cleaner is recommended for all Electric Attachments, Highspeed Handpieces, air motors and Scalers that require lubrication. Not Motors with sealed bearings