

INSTRUCTIONS FOR BIO LUBE E-FORMULA & ADVANCED DENTAL HANDPIECE LUBRICANT

LONGER LIFE • OZONE SAFE • BIODEGRADABLE • NON-TOXIC • ULTRA PURE

CODE: SOU01008
DENTAL HANDPIECE LUBRICANT



DANGER

EXTREMELY FLAMMABLE

MAY BE FATAL IF SWALLOWED AND ENTERS AIRWAYS

CONTENTS UNDER PRESSURE

KEEP OUT OF REACH OF CHILDREN

Read carefully other precautions on side panel.

PRECAUTIONS:

Store locked up. Dispose of contents/container to disposal recycling center. Under RCRA it is the responsibility of the user of the product to determine at the time of disposal whether the product meets RCRA criteria for hazardous waste. Waste management should be in full compliance with federal, state and local laws. Do not spray on an open flame or other ignition source. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. If medical advice is needed, have product container or label at hand. Keep out of reach of children. Read label before use.

First Aid:

IF SWALLOWED: Immediately call a POISON CENTER or doctor. Do NOT induce vomiting. Emergency phone number: Chemtrec US 1-800-424-9300, International 1-703-527-3887

SOUTHLAND DISTRIBUTION & SALES
4003 Enterprise Court
Augusta, GA 30907
1-800-880-0240
www.sdssouthland.com



Bio Lube™

with
TLC™
Technology

NEW *Advanced Synthetic Technology*

E-FORMULA

**Advanced Lubricant
For Electric Handpiece
Contra Angles
High Speed Handpieces**

Eliminates Friction • Runs Cooler • Quieter


It's Not Oil!

NET WT. 10 OZ. (283 g)

DIRECTIONS:

1. Flush & Clean Handpiece or Contra Angle with Bio Lube™ Advanced Cleaner.
2. Spray Bio Lube™ E-Formula into lubrication points (2-4 seconds). Follow manufacturer's instructions using proper adapters.
3. Run Handpiece 20-30 seconds to expel excess lubricant.
4. Wipe excess lubricant from Handpiece.
5. Bag & Sterilize Handpiece.
6. Remove from Sterilizer as soon as cycle ends; let cool.
7. Return Handpiece to service. (No need to lubricate after sterilization.)

*Use the red extension tube or **manufacturer's attachments.

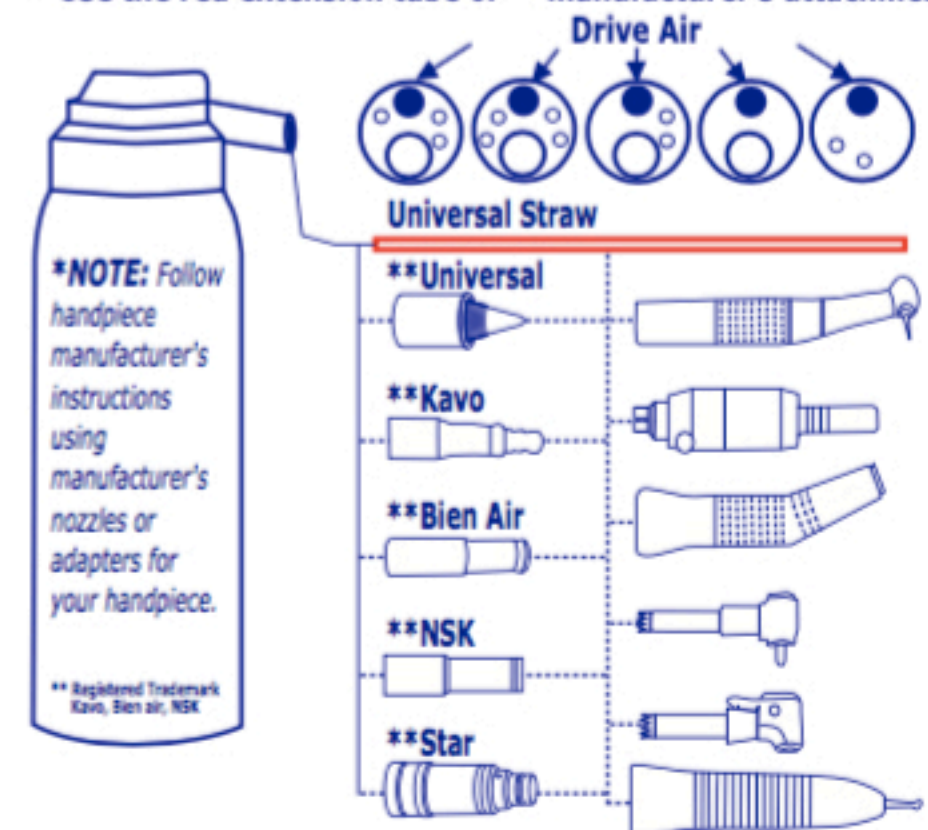

Bio Lube™ E-Formula lubricates all Electric Handpiece Contra angles, Air Motors, Scalers, and High Speed Handpieces.

REORDER
NUMBER **BIO-EF-500**

Read Safety Warnings on Side Label.