

BIODEGRADABLE NON-TOXIC NON-FLAMMABLE

Bio Lube™

Advanced Dental Handpiece Lubricant

with
TLC™ Technology

*High Temperature Lubricant
Extends Turbine Life
Runs Smooth, Quiet & Clean*



**BIODEGRADABLE
Synthetic Boundary Lubricant**

CAUTION: CONTENTS UNDER PRESSURE
Read Back Panel Carefully

**CODE: SOU01008
DENTAL HANDPIECE LUBRICANT**

Company Name: Southland Distribution & Sales
Street Address: 4003 ENTERPRISE COURT
City: AUGUSTA **State:** GA
Postal Code: 30907 **Country:** USA
Emergency Phone Number: CHEMTREC U.S.:
1-800-424-9300

INTERNATIONAL CALLS:
1-703-527-3887

Info Phone Number: 1-800-880-0240



WARNING

Hazardous Statement:

Pressurised Container: May burst if heated.

General:

If medical advice is needed, have product label at hand.
Keep out of reach of children. Read label before use.

Precautionary Statements:

Keep away from hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C / 122°F.

First Aid:

IF ON SKIN: Wash with plenty of water. If skin irritation occurs: Get medical advice/attention. Take off contaminated clothing and wash it before reuse. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention.

Ingestion Information:

Immediately call a POISON CENTER/doctor. Do not induce vomiting. If vomiting occurs naturally, lie on your side, in the recovery position. Do not give anything.

Contents 7 oz. (198.5 g) **BIO-L-091505-IBW**

STEP 1 - CLEAN

1. Holding handpiece in paper towel flush airline and exhaust line with **Bio Lube™** Cleaner for 2-3 seconds, expelling debris.



2. Repeat step 1 until no debris is visible.

STEP 2 - LUBRICATE

1. Spray **Bio Lube™** Lubricant into air line (short 1 second burst).
2. Run handpiece 15-20 seconds to expel excess lubricant.
3. Wipe excess lubricant from handpiece.
4. Bag & sterilize in Autoclave or Chemclave.
5. Remove handpiece from sterilizer and return to service.



NO LUBRICATION NEEDED AFTER STERILIZATION

Southland Distribution & Sales

www.sdssouthland.com

1-800-880-0240

info@sdssouthland.com

